*#402X*  
*Neoprene Chest Belt X*

*Positive*  
**Positioning**

## Table of Contents:

EN .....	4
NL .....	6
DE .....	8
FR .....	10
ES .....	12
IT .....	14
DA .....	16
NO .....	18
SK .....	20
FI .....	22
SV .....	24

## Neoprene Chest Belt X


### Application:

Positioning product to improve sitting posture. Please closely read and follow the instructions. Always use the chest positioning in combination with a hip belt! First unfasten the hip belt and only then the chest positioning!

### Warning:

This product is not a safety belt!

The product should be used only by authorised personnel and on their recommendation. The product's assembly and adjustments should be performed by authorised personnel.

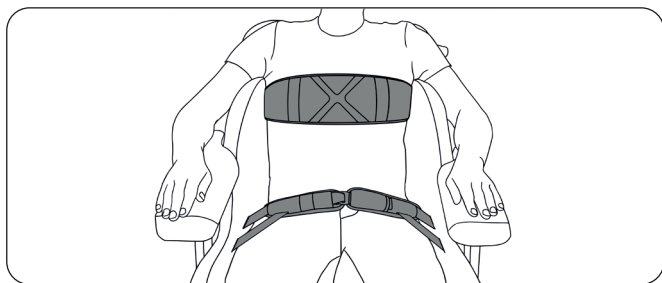### Maintenance:

Hand wash at a maximum of 30° Celsius. Do not tumble dry! Daily check the webbing straps' attachment and tension. Regularly check if the webbing straps, belts, and padded parts are worn out.

### Contents:

1. Tube Clamps
2. Webbing holders
3. Cam buckles
4. Ladder lock buckles
5. Positioning Belt
6. Set of instructions

2 x

2 x

2 x

2 x

## Neoprene Chest Belt X

### Assembly instructions:

1. Ensure that you have the right product in the right size.

Consideration: do not use the belt, if the client suffers from vascular disease or disrupted sensitivity. Seek advice from a doctor.

2. Carefully read the instructions included in the packaging.

3. Ensure the product is complete.

4. Adjust cam buckles' position.

Attention: not all chairs have attachment points.

Use the tube clamps if necessary. This product should be assembled by authorised personnel.

Consideration: the tube clamp is suitable for round tubes from 19 to 15 mm. Attach the clamp tube, whilst simultaneously attaching the cam buckle.

5. Adjust the webbing straps.

6. Avoid uncomfortable and alleviate pressure with regular intervals.






7. Always avoid constricting catheters and stomas.

8. Tuck away the webbing straps' end into the strap holders.

9. Ensure the webbing straps are not in contact with any of the wheelchair's moving/turning parts.

Attention: daily check the attachment and tension of the webbing straps. Regularly check if the webbing straps, belt, and padded parts are being worn out.

10. The side release buckles allow the chest positioning to be opened and fastened while being in use.

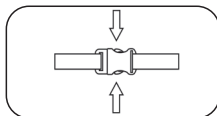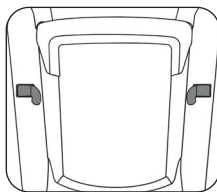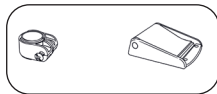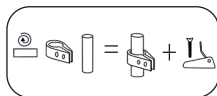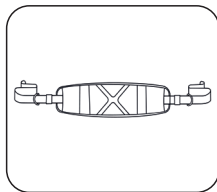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2222 AB Katwijk aan Zee