#5.81001.X
H-Chest Positioning
Flex-Control/Non-Stretch

Positive
Positioning

Table of Contents:

EN	4
NL	6
DE	8
FR	10
ES	12
IT	14
DA	16
NO	18
SK	20
FI	22
SV	24

H-Chest Positioning Flex-Control/Non-Stretch

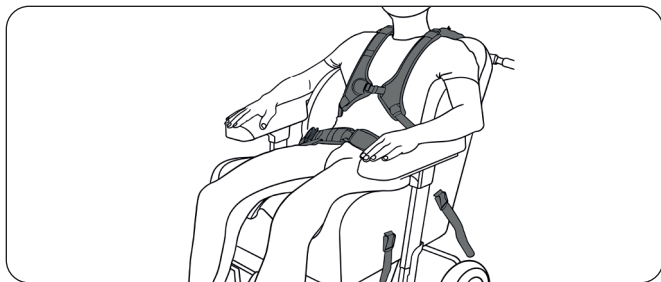
Application:

Positioning product to improve sitting posture. Please closely read and follow the instructions. Always use the chest positioning in combination with a hip belt! First unfasten the hip belt and only then unfasten the chest positioning vest!

Warning:

This product is not a safety belt!

The product should be used only by authorised personnel and on their recommendation. The product's assembly and adjustments should be performed by authorised personnel.

Maintenance:

Hand wash at a maximum of 30° Celsius. Do not tumble dry! Daily check the webbing straps' attachment and tension. Regularly check if the webbing straps, belts, and padded parts are worn out.

Contents:

1. Tube Clamps
2. Webbing holders
3. Cam buckles
4. Ladder lock buckles
5. Positioning Belt
6. Set of instructions

4 x

4 x

4 x

4 x

H-Chest Positioning Flex-Control/Non-Stretch

Assembly instructions:

1. Ensure you have the right product in the right size.

Consideration: do not use the belt if the client suffers from vascular disease or disrupted sensitivity. Seek advice from a doctor.

2. Carefully read the instructions included in the packaging.

3. Ensure the product is complete.

4. Adjust cam buckles' position.

Attention: not all chairs have attachment points.

Use the tube clamps if necessary. This product should be assembled by authorised personnel.

Consideration: the tube clamp is suitable for 19 to 15 mm round tubes. Attach the clamp tube, whilst simultaneously attaching the cam buckle.

5. Adjust the distance between the upper webbing straps. The paddings should rest against the shoulders but should not touch the neck.

6. Adjust the bottom webbing straps.

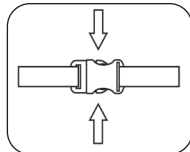
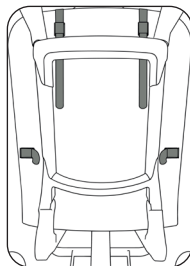
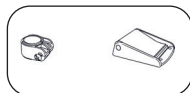
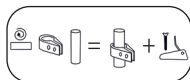
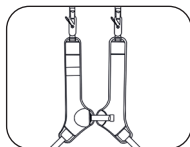
7. Avoid uncomfortable and alleviate pressure at regular intervals.

8. Tuck away the webbing straps' end into the strap holders.

9. Ensure the webbing straps are not in contact with any of the wheelchair's moving/turning parts.

Attention: daily check the attachment and tension of the webbing straps. Regularly check if the webbing straps, belts, and padded parts are worn out.

10. H-Chest Positioning can be opened and closed with the side release buckles. The middle fastener allows for adjusting the distance between the two padded parts. They can be opened/closed with a side release buckle.

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2222 AB Katwijk aan Zee