*#6.70045.X*  
*4-point Hip Belt Pin Release*  
*Double Adjustable*

*Positive*  
**Positioning**

## Table of Contents:

EN .....	4
NL .....	6
DE .....	8
FR .....	10
ES .....	12
IT .....	14
DA .....	16
NO .....	18
SK .....	20
FI .....	22
SV .....	24

## 4-point Hip Belt Pin Release Double Adjustable


### Application:

Positioning product to improve sitting posture. Please closely read and follow the instructions.

### Warning:

This product is not a safety belt!

The product should be used only by authorised personnel and on their recommendation. The product's assembly and adjustments should be performed by authorised personnel.

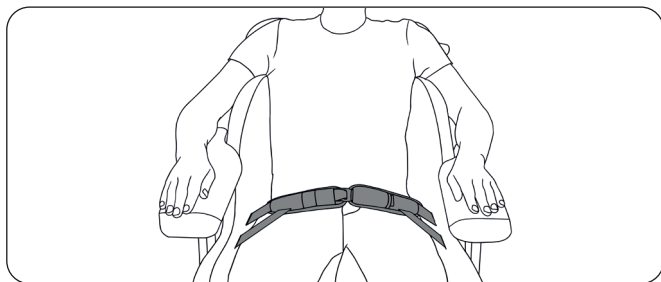### Maintenance:

Hand wash at a maximum of 30° Celsius. Do not tumble dry! Daily check the webbing straps' attachment and tension. Regularly check if the webbing straps, belts, and padded parts are worn out.

### Contents:

1. Tube Clamps
2. Webbing holders
3. Cam buckles
4. Ladder lock buckles
5. Positioning Belt
6. Set of instructions

4 x

4 x

4 x

4 x

## 4-point Hip Belt Pin Release Double Adjustable

### Assembly instructions:

1. Ensure you have the right product in the correct size.

Consideration: do not use the belt if the client suffers from vascular disease or disrupted sensitivity. Seek advice from a doctor.

2. Carefully read the instructions included in the packaging.

3. Ensure the product is complete.

4. Adjust cam buckles' position according to the belt's pull direction.

Attention: not all chairs have attachment points. Use the tube clamps if necessary. This product should be assembled by authorised personnel.

Consideration: the tube clamp is suitable for 19 to 15 mm round tubes. Attach the clamp tube, whilst simultaneously attaching the cam buckle.

5. When placing the client in the desired position, consider that the belt is lying loose on the sides of the wheelchair.

6. Close the belt.

7. Adjust the cam buckles to obtain the proper pressure distribution between the client and belt.

8. Avoid uncomfortable and alleviate pressure at regular intervals.






9. Always avoid constricting catheters and stomas.

10. Tuck away the webbing straps' end into the strap holders.

11. Ensure the webbing straps are not in contact with any of the wheelchair's moving/turning parts.

Attention: daily check the attachment and tension of the webbing straps. Regularly check if the webbing straps, belts, and padded parts are worn out.

12. Open the pin buckle by pressing it with a pen or bunt with a diameter smaller than 7 mm.

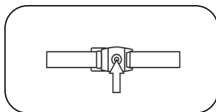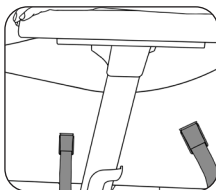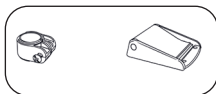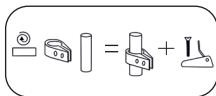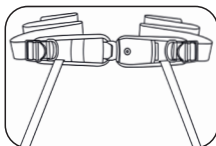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2222 AB Katwijk aan Zee