*#7.70002.X*  
*Pelvic Positioning Stretch*

## Table of Contents:

EN .....	4
NL .....	6
DE .....	8
FR .....	10
ES .....	12
IT .....	14
DA .....	16
NO .....	18
SK .....	20
FI .....	22
SV .....	24

## Pelvic Positioning Stretch


### Application:

Positioning product to improve posture. Please closely read and follow the instructions.

### Warning:

This product is not a safety belt!

The product should be used only by authorised personnel and on their recommendation. The product's assembly and adjustments should be performed by authorised personnel.

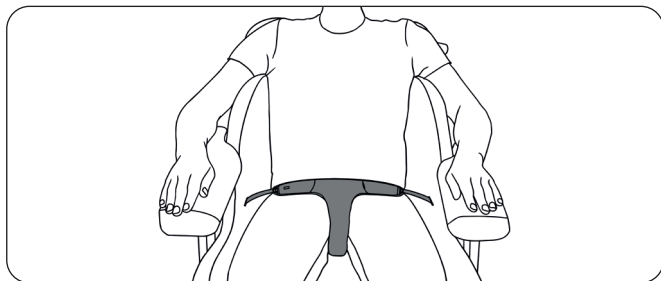### Maintenance:

Hand wash at a maximum of 30° Celsius. Do not tumble dry! Daily check the webbing straps' attachment and tension. Regularly check if the webbing straps, belts, and padded parts are worn out.

### Contents:

1. Tube Clamps
2. Webbing holders
3. Cam buckles
4. Ladder lock buckles
5. Positioning Belt
6. Set of instructions

3 x

3 x

3 x

3 x

## Pelvic Positioning Stretch

### Assembly instructions:

1. Ensure you have the right product in the correct size.  
Consideration: do not use the belt if the client suffers from vascular disease or disrupted sensitivity. Seek advice from a doctor.

2. Carefully read the instructions included in the packaging.

3. Ensure the product is complete.

4. When possible, remove the seating for easier assembly. After you determine the position of the cam buckles for webbing straps 1 and 2, attach webbing strap 3 to the centre of the chair's backside.

Attention: not all chairs have attachment points. Use the tube clamps if necessary. This product should be assembled by authorised personnel.

Consideration: the tube clamp is suitable for 19 to 15 mm round tubes. Attach the clamp tube, whilst simultaneously attaching the cam buckle.

5. Place the client in the chair in the desired position.

6. Position the product, slide webbing straps 1 and 2 through the cam buckles and fasten them in the correct pull direction.

7. Adjust webbing straps 1, 2 and 3.

8. Avoid uncomfortable and alleviate pressure at regular intervals.







9. Always avoid constricting catheters and stomas.

10. Tuck away the webbing straps' end into the strap holders.

11. Ensure the webbing straps are not in contact with any of the wheelchair's moving/turning parts.

Attention: daily check the attachment and tension of the webbing straps. Regularly check if the webbing straps, belts, and padded parts are worn out.

12. The side release buckles on webbing straps 1 and 2 allow the pelvic positioning to be opened and fastened while being in use.

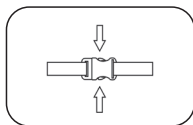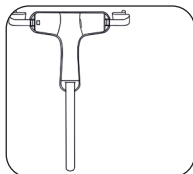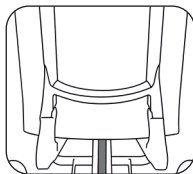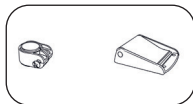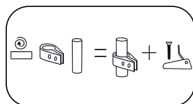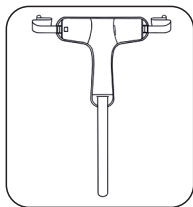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2222 AB Katwijk aan Zee