#8.70001.X
Ankle Positioning
Hook and Loop Closure

Table of Contents:

| | |
|----------|----|
| EN | 4 |
| NL | 6 |
| DE | 8 |
| FR | 10 |
| ES | 12 |
| IT | 14 |
| DA | 16 |
| NO | 18 |
| SK | 20 |
| FI | 22 |
| SV | 24 |

Ankle Positioning Hook and Loop Closure

Application:

Positioning product for the ankles. Please closely read and follow the instructions.



Warning:

This product is not a safety belt!

Always use the ankle positioning in combination with a hip belt!

First unfasten the ankle positioning and only then the hip belt. The product should be used only by authorised personnel and on their recommendation.

The product's assembly and adjustments should be performed by authorised personnel.

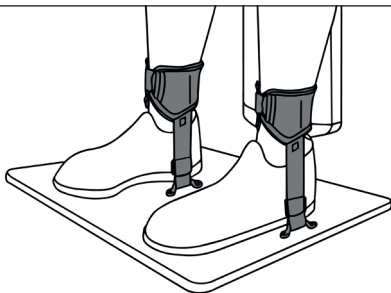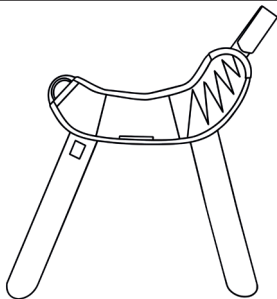Ankle Positioning Hook and Loop Closure

Assembly instructions:

1. Ensure you have the right product in the correct size.
Consideration: Do not use the belt if the client suffers from vascular disease or disrupted sensitivity. Seek advice from a doctor!
 2. Carefully read the instructions included in the packaging.
 3. Ensure the product is complete.
 4. Determine and then mark the most suitable position for the feet.
 5. Determine the proper position of the metal brackets and attach these to the footplate.
 6. Attach the ankle positioning.
 7. Fasten the ankle positioning using the hook and loop strips.
 8. Avoid uncomfortably and alleviate pressure at regular intervals.
 9. Always avoid constricting catheters and stomas.
 10. Tuck away the webbing straps' end into the strap holders.
 11. Ensure the webbing straps are not in contact with any of the wheelchair's moving/turning parts.
- Attention: Daily check the attachment and tension of the webbing straps. Regularly check if the webbing straps, belts, and padded parts are worn out.
12. Always use the ankle positioning in combination with a hip belt! First unfasten the ankle positioning and only then the hip belt.

Maintenance:

Hand wash at a maximum of 30° Celsius. Do not tumble dry! Daily check the webbing straps' attachment and tension. Regularly check if the webbing straps, belts, and padded parts are worn out.

Contents:

1. Attachment brackets
2. Ladder lock buckles
3. Positioning Belt
4. Set of instructions

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2222 AB Katwijk aan Zee