*#9.70001.X*  
*Abduction Positioning*

## Table of Contents:

EN .....	4
NL .....	6
DE .....	8
FR .....	10
ES .....	12
IT .....	14
DA .....	16
NO .....	18
SK .....	20
FI .....	22
SV .....	24

## Abduction Positioning


### Application:

Positioning product to improve sitting posture. Please closely read and follow the instructions.

### Warning:

This product is not a safety belt!

The product should be used only by authorised personnel and on their recommendation. The product's assembly and adjustments should be performed by authorised personnel.

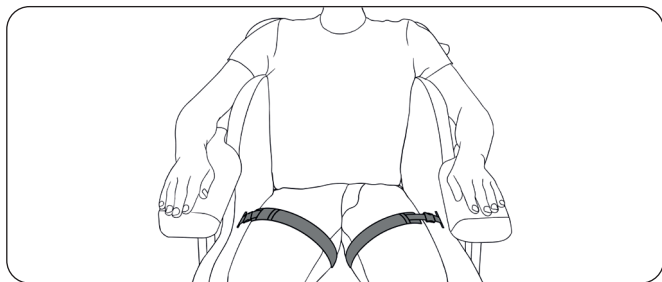### Maintenance:

Hand wash at a maximum of 30° Celsius. Do not tumble dry! Daily check the webbing straps' attachment and tension. Regularly check if the webbing straps, belts, and padded parts are worn out.

### Contents:

1. Tube Clamps
2. Webbing holders
3. Cam buckles
4. Ladder lock buckles
5. Positioning Belt
6. Set of instructions

4 x

4 x

4 x

4 x

## Abduction Positioning

### Assembly instructions:

1. Ensure you have the right product in the correct size.

Consideration: do not use the belt if the client suffers from vascular disease or disrupted sensitivity. Seek advice from a doctor!

2. Carefully read the instructions included in the packaging.

3. Ensure the product is complete.

4. Determine the position of the cam buckles on each side for webbing straps 1 and 2. Fasten webbing straps 3 and 4 on the backside of the chair by sliding them through the cam buckles.

Attention: not all chairs have attachment points. Use the tube clamps if necessary. This product should be assembled by authorised personnel.

Consideration: the tube clamp is suitable for 19 to 15 mm round tubes. Attach the clamp tube, whilst simultaneously attaching the cam buckle.

5. Place the product in the middle of the wheelchair seat. Place the client in the middle of the product, so that the ramus of the ischium is resting on middle of the seat.

6. Webbing straps 1 and 2 go through the cam buckles and should be fastened in the correct direction.

7. Adjust webbing straps 1 and 2 to avoid discomfort and alleviate pressure.






8. Always avoid constricting catheters and stomas.

Attention: daily check the attachment and tension of the webbing straps.

Regularly check if the webbing straps, belts, and padded parts are worn out.

9. Tuck away the webbing straps' end into the strap holders.

10. During use, the abduction positioning can be opened and fastened using the side release buckles on webbing straps 1 and 2.

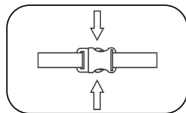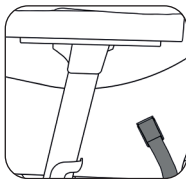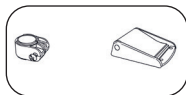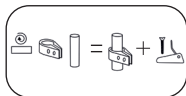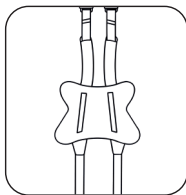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2222 AB Katwijk aan Zee