

PRODUCT RANGE





POSITIONING MATTERS

At Positioning Products, we believe that everyone deserves to feel good in their body, live a healthy life, and enjoy comfort. Therefore, by paying attention to the details and listening to our clients and users, we design durable, high-quality, good-looking belts and harnesses that improve the health of wheelchair users.

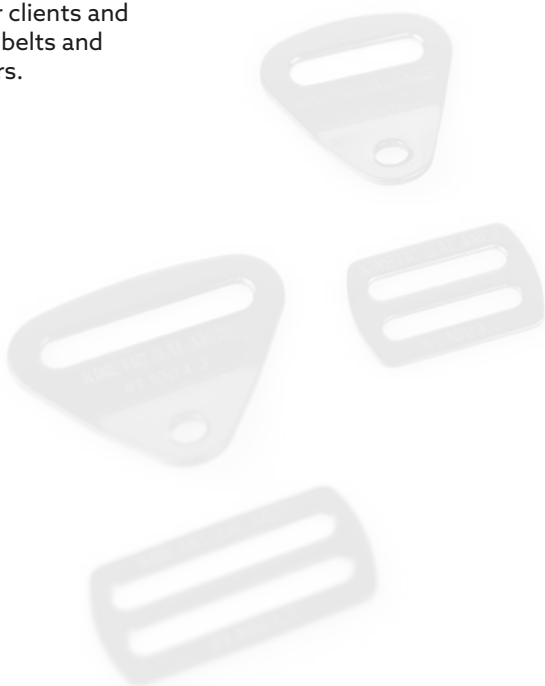


TABLE OF CONTENTS

+ KINETIC BALANCE POSITIONING PRODUCTS	4
+ CHARACTERISTICS	5
+ HIP POSITIONING	6
Buckles and Adjustments	7
Hip Belts	8
How to measure	8
2-point hip belts	8
4-point hip belts	10
Pelvic Belts	12
How to measure	12
T-shape belts	13
+ CHEST POSITIONING	14
Chest Harnesses	14
How to measure	14
Stretch	15
Non-stretch	15
Chest Belts	16
How to measure	16
Chest belt X	17
Chest belt V1	17
+ LOWER LIMBS POSITIONING	18
Ankle Positioning	18
How to measure	18
+ ABDUCTION POSITIONING	20
How to measure	20
+ ASSORTED POSITIONING	22
How to measure	22
Shower Belt	23
Multi Purpose Belt	23
+ SPARE PARTS	24
+ OVERVIEW & ARTICLE NUMBERS	26
+ PRODUCT PRESENTATION	30
+ ISO 9001:2015	31

EMPOWERING LIVES

At Positioning Products, we are committed to enhancing the lives of wheelchair users through thoughtfully designed positioning solutions.

Our products are developed to support a healthy, functional seating posture by providing anterior postural support. A good posture helps conserve energy throughout the day and contributes to overall wellbeing.

We believe belts, harnesses and other positioning products are essential components of a seating system. Rather than using energy to maintain posture, users should be able to focus on their daily activities. Our positioning products help accommodate or correct postural deviations, promoting comfort, stability and improved quality of life.

PRODUCT CHARACTERISTICS

QUALITY MATERIALS. FUNCTIONAL DESIGNS. SAFE USE.

At Positioning Products, our belts and harnesses are built for durability, safety and ease of use. Designed with attention to every detail and a focus on aesthetic appeal. Our dedicated team understands that product quality directly impacts user safety. That's why components are carefully considered; Buckles for example are a key element in all our products, most of them are covered with a pocket to protect the user from their hard edges.

The harnesses have a grip print on their inner side to secure the product's correct position on the person's body and prevent unintentional movements. All Positioning Products are made from water and dirt repellent materials, ensuring longevity. When it comes to easy of use, it is essential to highlight that the webbings on our products have rounded edges and a trim strip, guaranteeing their seamless and precise mounting on any wheelchair.



+ WATER AND DIRT REPELLENT



+ PREMIUM QUALITY MATERIALS



+ COVERED FASTENERS



+ MATERIAL REINFORCEMENTS



+ SUPER-DURABLE STITCHING



+ GRIP PRINT



+ CLEAR SIZE INDICATION



+ CE MARKED PRODUCT



+ QUALITY CONTROL



+ TRIMSTRIP



+ ROUNDED WEBBINGS



+ CERTIFIED








HAVING A GOOD SEATING POSTURE STARTS WITH POSITIONING THE PELVIS, WHICH IS THE BASIS OF SEATING

Incorrect seating posture can negatively affect a person's body, and could lead to pain and poor circulation, digestion, and breathing.

The initial step to good posture is to properly fit the wheelchair to the user's needs, which requires providing support under the sitting surface. This can be achieved by matching the size of the seat, matching the correct height and width of the backrest and by matching leg-, arm- and if needed head support.

However, sometimes, the wheelchair alone cannot solve users' postural deviation. Therefore, we have designed a wide range belts and harnesses, which can further assist in positioning the user.

- Our Hip Belts come with various buckles and are either single or double-adjustable. The Hip Belts tackle problems like:
- + Posterior Pelvic Tilt
 - + Anterior Pelvic Tilt
 - + Pelvic Rotation
 - + Pelvic Obliquity

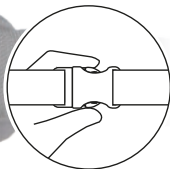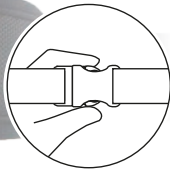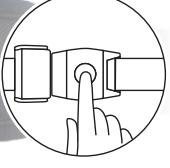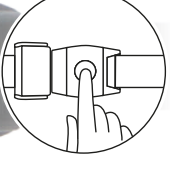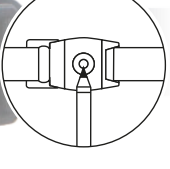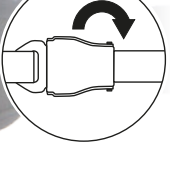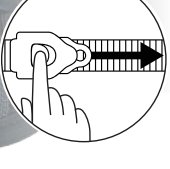		SIDE RELEASE SINGLE ADJUSTABLE
		SIDE RELEASE DOUBLE ADJUSTABLE
		CENTER RELEASE SINGLE ADJUSTABLE
		CENTER RELEASE DOUBLE ADJUSTABLE
		PIN RELEASE DOUBLE ADJUSTABLE
		AVIATION RELEASE SINGLE ADJUSTABLE
		RATCHET RELEASE DOUBLE ADJUSTABLE

HIP POSITIONING 2-POINT HIP BELTS



HOW TO MEASURE

POSTERIOR PELVIC TILT


The distance from the lateral side of one thigh to the lateral side of the other thigh shall be measured with a flexible tape measure.

ANTERIOR PELVIC TILT


For a support managing anterior pelvic tilt, the distance from one lateral side of the pelvis horizontally level with the ASISs to the other lateral side at the same level shall be measured with a flexible tape measure.

SIZE CHART

Size	Length (mm)	
XXS	220 - 270	230 - 280
XS	360 - 410	310 - 360
S	360 - 410	380 - 430
M	450 - 500	450 - 510
L	570 - 620	510 - 560
XL	670 - 720	570 - 620
	6.70000.X-6.70006.X	6.70007.X

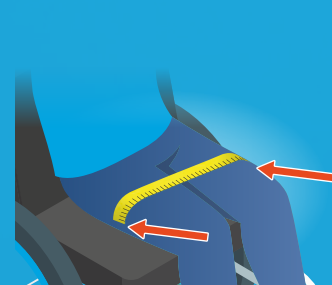

POSTERIOR PELVIC TILT

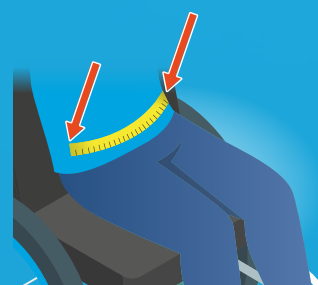


ANTERIOR PELVIC TILT

2-POINT HIP BELTS


2-POINT ESSENTIAL HIP BELT SINGLE ADJUSTABLE

Size: M
Article number: 6.70000.2
Made to solve Posterior Pelvic Tilt

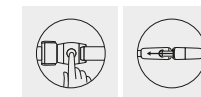


2-POINT HIP BELT SIDE RELEASE SINGLE ADJUSTABLE

Size: XXS/XS/S/M/L/XL
Article number: 6.70002.x
Made to solve Posterior Pelvic Tilt

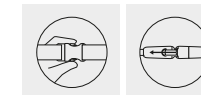


2-POINT HIP BELT CENTRE RELEASE DOUBLE ADJUSTABLE

Size: XS/S/M/L/XL
Article number: 6.70004.x
Made to solve Posterior Pelvic Tilt & Pelvic Obliquity

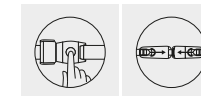


2-POINT HIP BELT AVIATION RELEASE SINGLE ADJUSTABLE

Size: S/M/L/XL
Article number: 6.70006.x
Made to solve Posterior Pelvic Tilt

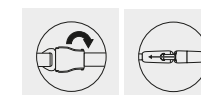


2-POINT HIP BELT SIDE RELEASE DOUBLE ADJUSTABLE

Size: XS/S/M/L/XL
Article number: 6.70001.x
Made to solve Posterior Pelvic Tilt & Pelvic Obliquity

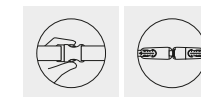


2-POINT HIP BELT CENTRE RELEASE SINGLE ADJUSTABLE

Size: XS/S/M/L/XL
Article number: 6.70003.x
Made to solve Posterior Pelvic Tilt

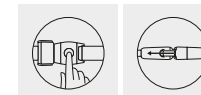


2-POINT HIP BELT PIN RELEASE DOUBLE ADJUSTABLE

Size: XS/S/M/L/XL
Article number: 6.70005.x
Made to solve Posterior Pelvic Tilt & Pelvic Obliquity

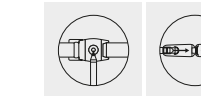

2-POINT HIP BELT RATCHET RELEASE SINGLE ADJUSTABLE

Size: XXS/XS/S/M/L/XL
Article number: 6.70007.x
Made to solve Posterior Pelvic Tilt

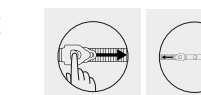

HIP POSITIONING

4-POINT


HIP BELTS



4-POINT HIP BELTS


4-POINT HIP BELT SIDE RELEASE SINGLE ADJUSTABLE

Size: XS/S/M/L/XL
Article number: 6.70042.x
Made to solve Anterior Pelvic Tilt & Pelvic Rotation

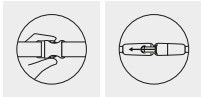


4-POINT HIP BELT SIDE RELEASE DOUBLE ADJUSTABLE

Size: XS/S/M/L/XL
Article number: 6.70044.x
Made to solve Anterior Pelvic Tilt & Pelvic Rotation

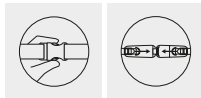


4-POINT HIP BELT CENTRE RELEASE SINGLE ADJUSTABLE

Size: XS/S/M/L/XL
Article number: 6.70047.x
Made to solve Anterior Pelvic Tilt & Pelvic Rotation

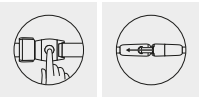


4-POINT HIP BELT CENTRE RELEASE DOUBLE ADJUSTABLE

Size: XS/S/M/L/XL
Article number: 6.70043.x
Made to solve Anterior Pelvic Tilt & Pelvic Rotation

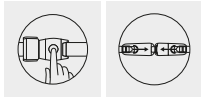

4-POINT HIP BELT PIN RELEASE DOUBLE ADJUSTABLE

Size: XS/S/M/L/XL
Article number: 6.70045.x
Made to solve Posterior Pelvic Rotation

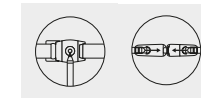

HOW TO MEASURE

POSTERIOR PELVIC TILT


The distance from the lateral side of one thigh to the lateral side of the other thigh shall be measured with a flexible tape measure.

ANTERIOR PELVIC TILT

For a support managing anterior pelvic tilt, the distance from one lateral side of the pelvis horizontally level with the ASISs to the other lateral side at the same level shall be measured with a flexible tape measure.

SIZE CHART


Size	Length (mm)
XS	360 - 410
S	360 - 410
M	450 - 500
L	570 - 620
XL	670 - 720

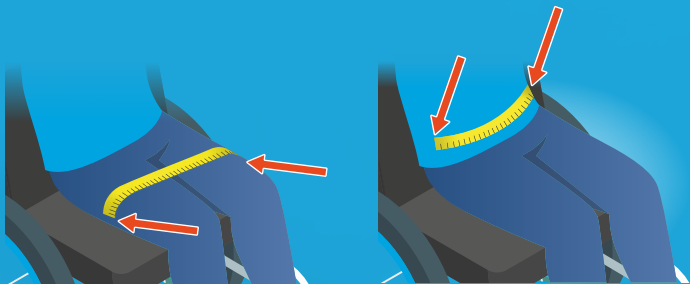

POSTERIOR PELVIC TILT

ANTERIOR PELVIC TILT

HIP POSITIONING PELVIC BELTS

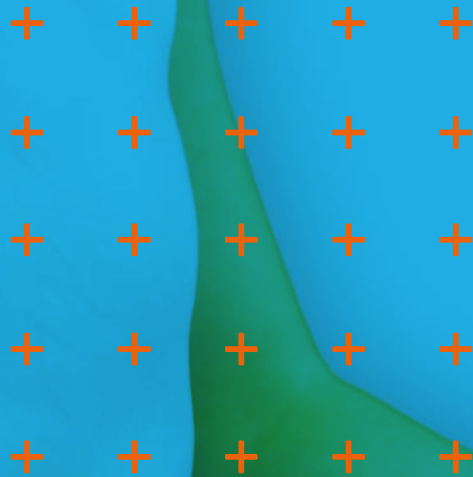


HOW TO MEASURE

POSTERIOR PELVIC TILT & ANTERIOR PELVIC TILT
The position of the pelvis should be assessed by measuring the distance between the two Anterior Superior Iliac Spines (ASIS).

◆ SIZE CHART

Size	Length (mm)
S	135 - 185
M	190 - 240
L	270 - 320
XL	370 - 420

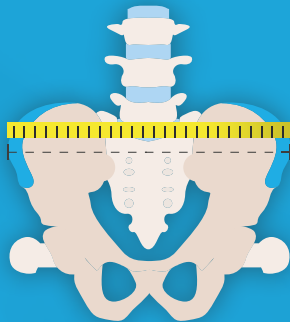

T-SHAPE PELVIC BELTS

**PELVIC POSITIONING
NON-STRETCH**
Size: S/M/L/XL
Article number: 7.70001.x
Made to solve Anterior Pelvic Tilt & Pelvic Rotation



**PELVIC POSITIONING
STRETCH**
Size: S/M/L/XL
Article number: 7.70002.x
Made to solve Anterior Pelvic Tilt & Pelvic Rotation



CHEST POSITIONING HARNESES




HOW TO MEASURE

ALWAYS MUST BE USED IN COMBINATION WITH A HIP BELT

ANTERIOR TRUNK SUPPORT
The shoulder width shall be measured as shown.

SIZE CHART

Size	Length (mm)
XS	150 - 240
Junior	150 - 240
S	240 - 280
M	280 - 330
L	330 - 410
XL	410 - 480

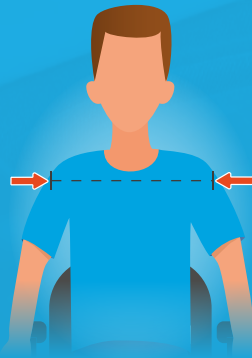

CHEST POSITIONING STRETCH & NON-STRETCH

CHEST POSITIONING STRETCH

Size: XS/Junior/S/M/L
Article number Junior: 5.71001.1
Article number: 5.71012.x
Made to solve Kyphosis & Lordosis



X-CHEST POSITIONING STRETCH

Size: S/M/L
Article number: 5.71014.x
Made to solve Kyphosis & Lordosis



CHEST POSITIONING ZIPPER STRETCH

Size: S/M
Article number: Z5.71022.x
Made to solve Kyphosis & Lordosis



CHEST POSITIONING FLEXCONTROL/NON-STRETCH

Size: S/M/L/XL
Article number: 5.71032.x
Made to solve Kyphosis & Lordosis



X-CHEST POSITIONING FLEXCONTROL/NON-STRETCH

Size: S/M/L
Article number: 5.71034.x
Made to solve Kyphosis & Lordosis



H-CHEST POSITIONING FLEX-CONTROL/ NON-STRETCH

Size: S/M/L/XL
Article number: 5.81001.x
Made to solve Kyphosis & Lordosis



CHEST POSITIONING CHEST BELTS




HOW TO MEASURE

The circumferential distance shall be measured with a flexible tape, around the torso to the back support on each side.

SIZE CHART

Size	Length (mm)
XS	310 - 410
S	400 - 500
M	470 - 570
L	550 - 650

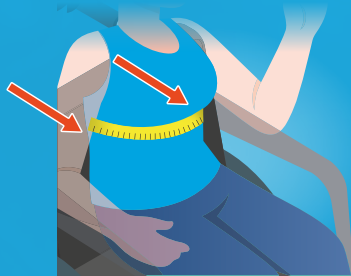

CHEST BELTS

NEOPRENE CHEST BELT X

Size: XS/S/M/L
Article Number: 402x
Made to solve Lordosis



CHEST BELT V1

Size: S/L
Article Number: 400x
Made to solve Lordosis



LOWER LIMBS POSITIONING

ANKLE POSITIONING



HOW TO MEASURE

ALWAYS MUST BE USED IN COMBINATION WITH A HIP BELT

The circumference above the ankle bone shall be measured with a flexible tape measure.

The circumference shall be measured with the user wearing shoes and ankle foot orthoses (AFOs) if these are typically worn in the seating system.

SIZE CHART

Size	Length (mm)
XS	190 - 220
S	220 - 250
M	270 - 300
L	300 - 330



ANKLE POSITIONING

ANKLE POSITIONING HOOK AND LOOP CLOSURE

Size: XS/S/M/L
 Article number: 8.70001.x
 Made to solve Feet going of footrest &
 Correcting the position of the feet on
 the footrest



ANKLE POSITIONING SIDE RELEASE

Size: XS/S/M/L
 Article number: 8.70002.x
 Made to solve Feet going of footrest &
 Correcting the position of the feet on
 the footrest



ABDUCTION POSITIONING




HOW TO MEASURE

Measure from the lateral side of the thigh to the point where the inner side of the thigh touches the seat.

SIZE CHART

Size	Length (mm)
XS	140 - 190
S	170 - 220
M	200 - 250
L	250 - 300

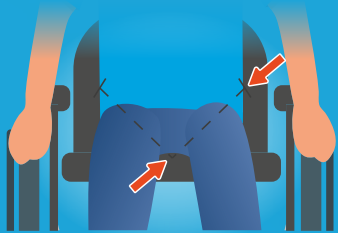

ABDUCTION POSITIONING

ABDUCTION POSITIONING
Size: XS/S/M/L
Article Number: 9.70001.x
Made to solve Adduction of the upper legs




ASSORTED POSITIONING



HOW TO MEASURE

SHOWER BELT
Measure from the top of seat to top of seat.

Size	Length (mm)
S	400 - 710
M	600 - 910
L	740 - 1070

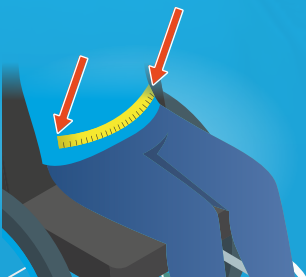

NEOPRENE MULTI-PURPOSE BAND*
Measure from the fixation points on the chair frame, wrapping behind the occupant's lower legs using a flexible tape measure.

Size	Length (mm)
S	650 - 900
M	750 - 1000
L	950 - 1200




* Depending on use

ASSORTED POSITIONING

SHOWER BELT
Size: S/M/L
Article number: 300x
Positioning aid for in the shower



NEOPRENE MULTI-PURPOSE BAND
Size: S/M/L
Article Number: 500x

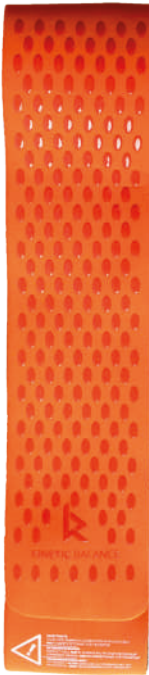

+ SPARE PARTS

Article #	Article Name	
1.1001.1	10 pcs Webbing Holder 20mm	
1.1001.2	10 pcs Webbing Holder 25mm	
1.1001.3	10 pcs Webbing Holder 40mm	
1.1002.1	10 pcs Slider Buckle 20mm	
1.1002.2	10 pcs Slider Buckle 25mm	
1.1002.3	10 pcs Slider Buckle 40mm	
1.1003.2	10 pcs Metal Slider Buckle 25mm	
1.1003.3	10 pcs Metal Slider Buckle 40mm	
1.1004.2	10 pcs Metal Triangle Bracket 25mm	
1.1004.3	10 pcs Metal Triangle Bracket 40mm	
1.1005.0	10 pcs Tube Clamps	
1.1006.2	10 pcs Centre Release Plastic Cap 25 mm	
1.1006.3	10 pcs Centre Release Plastic Cap 40 mm	
1.1007.1	10 pcs D-Ring 20mm	
1.1007.2	10 pcs D-Ring 25mm	
1.1007.3	10 pcs D-Ring 40mm	

+ RATCHET BELT ITEMS

Article #	Article Name	
1.3001.1	10 pcs Ratchet Buckle - GO - 14	
1.3001.2	10 pcs Ratchet Buckle - ERGO - 19	
1.3001.3	10 pcs Ratchet Buckle - ERGO - 24	
1.3002.1	10 pcs Ratchet V-Straps - GO - 14	
1.3002.2	10 pcs Ratchet V-Straps - ERGO - 19	
1.3002.3	10 pcs Ratchet V-Straps - ERGO - 24	

+ BUCKLES

Article #	Article Name	
1.2001.1 1.2001.2 1.2001.3	10 pcs Side Release Buckle 20mm 10 pcs Side Release Buckle 25mm 10 pcs Side Release Buckle 40mm	
1.2002.2 1.2002.3	10 pcs Centre Release Buckle 25mm 10 pcs Centre Release Buckle 40mm	
1.2003.2 1.2003.3	10 pcs Centre Release Buckle 25mm with cap 10 pcs Centre Release Buckle 40mm with cap	
1.2004.2 1.2004.3	10 pcs Pin Release Buckle 25mm 10 pcs Pin Release Buckle 40mm	
1.2005.3	10 pcs Aviation Release Buckle 40mm	
1.2006.2 1.2006.3	10 pcs Cam buckle 25mm 10 pcs Cam Buckle 40mm	

+ SETS

Article #	Article Name	
1.4001.1	4 pcs Straps for Chest Positionig Webbing Straps 20 mm	
1.4001.2	4 pcs Straps for Chest Positionig Webbing Straps 25 mm	
1.4002.2	Ankle Positioning Mounting Kit 25mm	
1.4002.3	Ankle Positioning Mounting Kit 40mm	
1.4003.2	10 pcs Side Release Contrabelts 25mm	
1.4003.3	10 pcs Side Release Contrabelts 40mm	

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

PRODUCT OVERVIEW

+ HIP | 2-POINT

2-point Esesntial Hip Belt Single Adjustable		
6.70000.2	M	

2-point Hip Belt Side Release Double Adjustable		
6.70001.0	XS	
6.70001.1	S	
6.70001.2	M	
6.70001.3	L	
6.70001.4	XL	

2-point Hip Belt Side Release Single Adjustable		
6.70002.00	XXS	
6.70002.0	XS	
6.70002.1	S	
6.70002.2	M	
6.70002.3	L	
6.70002.4	XL	

2-point Hip Belt Centre Release Single Adjustable		
6.70003.0	XS	
6.70003.1	S	
6.70003.2	M	
6.70003.3	L	
6.70003.4	XL	


2-point Hip Belt Centre Release Double Adjustable		
6.70004.0	XS	
6.70004.1	S	
6.70004.2	M	
6.70004.3	L	
6.70004.4	XL	

2-point Hip Belt Pin Release Double Adjustable		
6.70005.0	XS	
6.70005.1	S	
6.70005.2	M	
6.70005.3	L	
6.70005.4	XL	

2-point Hip Belt Aviation Release Single Adjustable		
6.70006.1	S	
6.70006.2	M	
6.70006.3	L	
6.70006.4	XL	

2-point Hip Belt Ratchet Release Single Adjustable		
6.70007.00	XXS	
6.70007.0	XS	
6.70007.1	S	
6.70007.2	M	
6.70007.3	L	
6.70007.4	XL	

Measurements
2-point Hip Belt

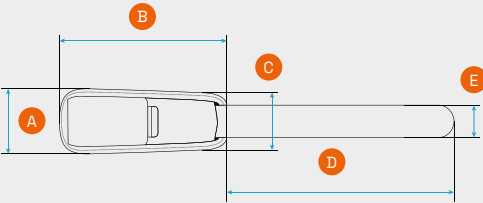

Article 6.70000 - 6.70006

	A	B	C	D	E
XXS	50	110	40	480	20
XS	60	180	45	580	25
S	70	180	55	640	38
M	80	225	60	720	38
L	85	285	65	780	38
XL	100	335	70	780	38

All measurements in mm

Article 6.70007

	A	B	C	D	E
XXS	65	110	53	500	25
XS	65	150	53	500	25
S	70	190	58	675	38
M	70	225	58	675	38
L	75	255	60	700	38
XL	80	285	60	700	38

All measurements in mm

+ HIP | 4-POINT

4-point Hip Belt Side Release Single Adjustable		
6.70042.0	XS	
6.70042.1	S	
6.70042.2	M	
6.70042.3	L	
6.70042.4	XL	


4-point Hip Belt Side Release Double Adjustable		
6.70044.0	XS	
6.70044.1	S	
6.70044.2	M	
6.70044.3	L	
6.70044.4	XL	

4-point Hip Belt Centre Release Single Adjustable		
6.70047.0	XS	
6.70047.1	S	
6.70047.2	M	
6.70047.3	L	
6.70047.4	XL	

4-point Hip Belt Centre Release Double Adjustable		
6.70043.0	XS	
6.70043.1	S	
6.70043.2	M	
6.70043.3	L	
6.70043.4	XL	

4-point Hip Belt Pin Release Double Adjustable		
6.70045.0	XS	
6.70045.1	S	
6.70045.2	M	
6.70045.3	L	
6.70045.4	XL	

Measurements
4-point Hip Belt

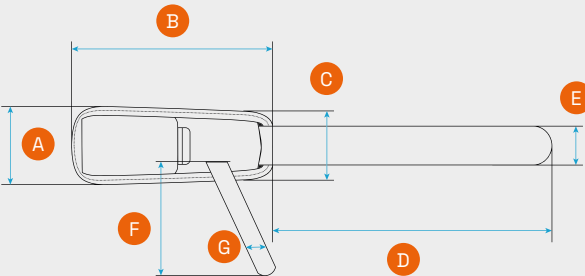

	A	B	C	D	E	F	G
XS	60	180	45	580	25	500	20
S	70	180	55	640	38	500	25
M	80	225	60	720	38	600	25
L	85	285	65	780	38	700	25
XL	100	335	70	780	38	700	25

All measurements in mm


+ CHEST HARNESSES | STRETCH

Chest Positioning Stretch		
5.71012.0	XS	
5.71001.1	Junior	
5.71012.1	S	
5.71012.2	M	
5.71012.3	L	

X-Chest Positioning Stretch		
5.71014.1	S	
5.71014.2	M	
5.71014.3	L	

Chest Positioning Zipper Stretch		
Z5.71022.1	S	
Z5.71022.2	M	


Measurements
Chest Positioning
Stretch

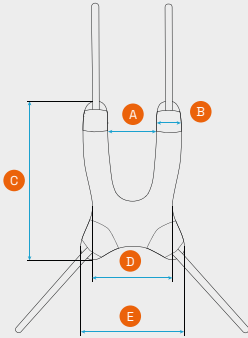

	A	B	C	D	E
XS	50	50	230	80	170
Junior	75	55	265	150	195
S	70	55	310	160	215
M	110	60	350	180	230
L	120	60	395	200	245

All measurements in mm


Measurements
X-Chest Positioning
Stretch

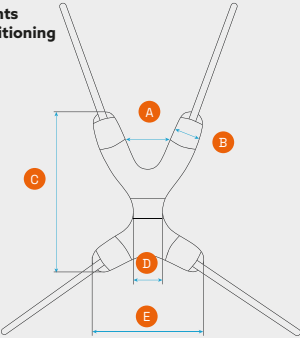

	A	B	C	D	E
S	80	60	310	75	230
M	90	60	360	75	250
L	120	65	410	75	285

All measurements in mm

Measurements
Chest Positioning Zipper
Stretch

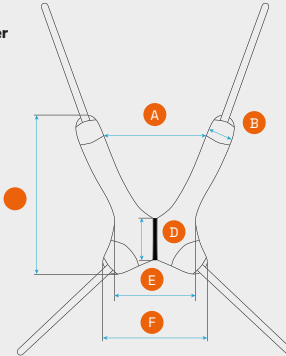

	A	B	C	D	E	F
S	115	55	305	85	150	190
M	180	60	350	85	180	225

All measurements in mm


+ CHEST HARNESSES | NON-STRETCH

Chest Positioning Flex-Control/Non-Stretch		
5.71032.1	S	
5.71032.2	M	
5.71032.3	L	
5.71032.4	XL	

X-Chest Positioning Flex-Control/Non-Stretch		
5.71034.1	S	
5.71034.2	M	
5.71034.3	L	

H-Chest Positioning Flex-Control/Non-Stretch		
5.81001.1	S	
5.81001.2	M	
5.81001.3	L	
5.81001.4	XL	


Measurements
Chest Positioning
Flex-control/
Non-Stretch

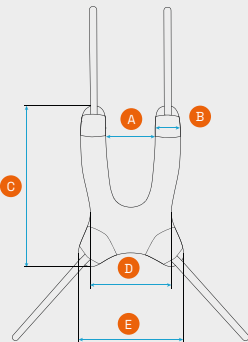

	A	B	C	D	E
S	70	55	310	160	215
M	110	60	350	180	230
L	120	60	395	200	245
XL	160	60	450	235	290

All measurements in mm


Measurements
X-Chest Positioning
Flex-control/
Non-Stretch

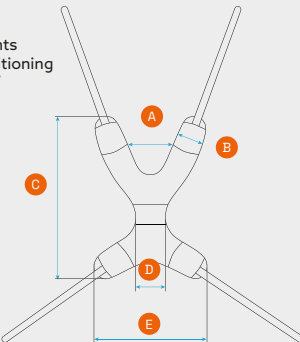

	A	B	C	D	E
S	80	60	310	75	230
M	90	60	360	75	250
L	120	65	410	75	285

All measurements in mm

Measurements
H-Chest Positioning
Flex-control/
Non-Stretch

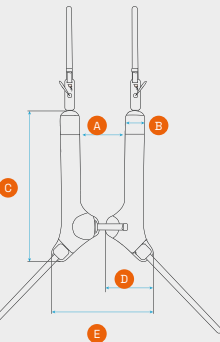

	A	B	C	D	E
S	70	50	310	100	200
M	100	55	390	100	235
L	140	55	470	120	280
XL	150	70	530	140	310

All measurements in mm

PRODUCT OVERVIEW


+ CHEST BELTS

Neoprene Chest Belt X		
4020	XS	
4021	S	
4022	M	
4023	L	

Measurements
Neoprene Chest Belt X

	A	B	C
XS	305	80	25
S	400	100	38
M	460	110	38
L	565	130	38

All measurements in mm

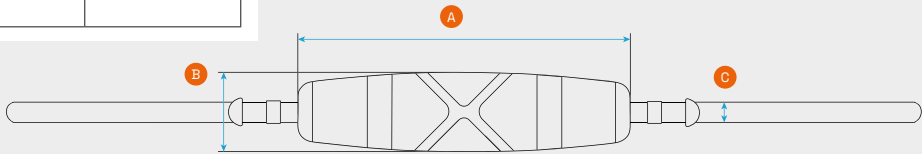


Chest Belt V1		
4001	S	
4002	L	

Measurements
Chest Belt V1

	A	B
S	1400>1600	50
M	1700>1950	100

All measurements in mm

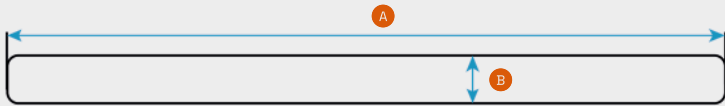


+ ASSORTED POSITIONING

Neoprene Multi-Purpose Band		
5001	S	
5002	M	
5003	L	

Measurements
Neoprene Multi-Purpose Band

	A	B
S	1100	90
M	1300	100
L	1500	140

All measurements in mm

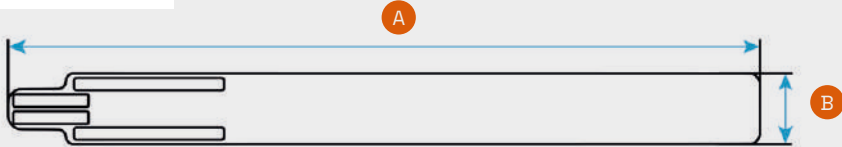


Shower Belt		
3001	S	
3002	M	
3003	L	

Measurements
Shower Belt

	A	B	C
S	315	105	38
M	415	105	38
L	490	105	38

All measurements in mm

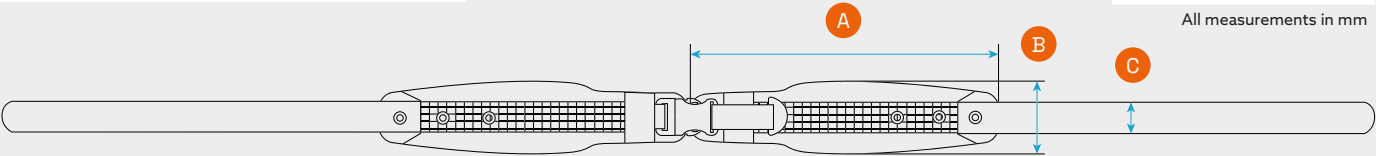

+ PELVIC

Pelvic Positioning Non-Stretch		
7.70001.1	S	
7.70001.2	M	
7.70001.3	L	
7.70001.4	XL	

Pelvic Positioning Stretch		
7.70002.1	S	
7.70002.2	M	
7.70002.3	L	
7.70002.4	XL	


+ ANKLE

Ankle Positioning Hook and Loop Closure		
8.70001.0	XS	
8.70001.1	S	
8.70001.2	M	
8.70001.3	L	


Ankle Positioning Side Release		
8.70002.0	XS	
8.70002.1	S	
8.70002.2	M	
8.70002.3	L	

+ ABDUCTION

Abduction Positioning		
9.70001.0	XS	
9.70001.1	S	
9.70001.2	M	
9.70001.3	L	

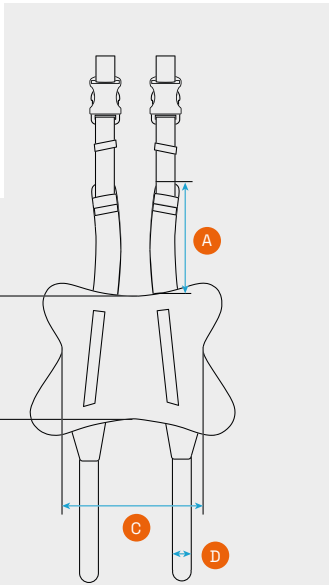


Measurements
Pelvic Positioning

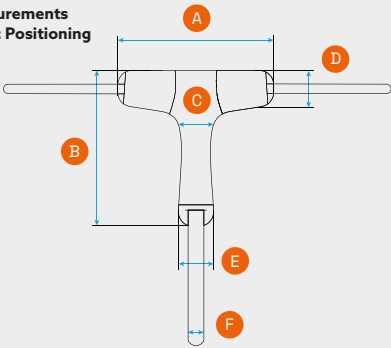

	A	B	C	D	E	F
S	185	190	155	55	50	20
M	230	230	265	65	55	25
L	310	310	370	70	65	38
XL	410	410	490	80	85	38

All measurements in mm

Measurements
Ankle Positioning

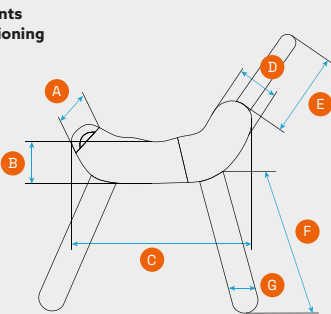

	A	B	C	D	E	F	G
XS	45	50	190	55	90	270	20
S	50	60	220	65	140	280	25
M	60	65	270	65	140	350	38
L	60	65	300	65	170	365	38

All measurements in mm

Measurements
Abduction Positioning


	A	B	C	D
XS	140	150	190	25
S	170	180	220	25
M	200	210	240	25
L	250	240	270	38

All measurements in mm

INTERESTED IN OUR POSITIONING PRODUCTS?

Would you like to learn more about our positioning products and how they can improve seating posture? We'd be happy to share more information. If interested, a product specialist can visit you to demonstrate the benefits and possibilities of our products and discuss important considerations for mounting and placement.

We're look forward to exploring the possibilities together.

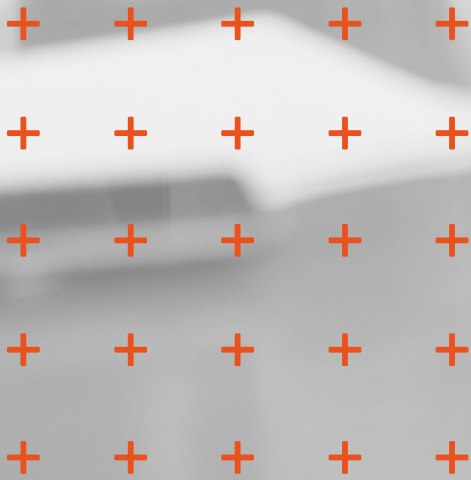

DEDICATED TO DELIVERING TOP-QUALITY

ISO 9001 is the international standard that specifies requirements for a quality management system (QMS). Organizations use the standard to demonstrate the ability to consistently provide products and services that meet customer and regulatory requirements. Our manufacturing facility is compliant with this standard. Thus, ensuring the superior quality of all our products. We promise to continue to provide you with the best positioning products available.



Photo by Gelbart Orto Team



Positioning Products B.V.
Scheepmakerstraat 27
2222 AB Katwijk aan Zee
The Netherlands

T: +31 (0)71-2048424
E: mail@positioningproducts.nl

